

MANITOBA WHEELCHAIR PROGRAM – QUICK REFERENCE GUIDE

MWP CATEGORY	TYPE OF USER / INTENDED USE	WHEELCHAIR FEATURES	WHEELCHAIR MODEL
<h3>Category 1A / 1B</h3>	<ul style="list-style-type: none"> • Transport Chair • Part time use • Can meet the majority of their functional mobility needs via ambulation • Long distances only • Transfers to a vehicle seat for transportation • Does not require extensive supports • No positioning concerns 	<p>Seat Width Options: 1A: 16", 18", 20" 1B: 20", 22", 24"</p> <p>Seat Depth Options: 16" 18" (for 22" and 24" widths only)</p> <p>Seat Height Options: 17.75" 19.75"</p> <ul style="list-style-type: none"> • Fixed horizontal axle, two vertical positions • Solid Tires • Fixed height, removable armrests 	<p>Breezy Easy Care 2000</p> <p>Weight Capacity: 250lbs (16"W, 18"W)</p> <p>Breezy Easy Care 2000HD</p> <p>Weight Capacity: 300lbs (20" W) 450lbs (22"W, 24"W)</p>
<h3>Category 2A</h3>	<ul style="list-style-type: none"> • Transport Chair • Up to full time use • May or may not transfer to a vehicle seat • May or may not be able to self propel functional distances • May require seating • May present with some positioning needs to allow functional use of the mobility base 	<p>Seat Width Options: 16", 18", 20"</p> <p>Seat Depth: 16", 18"</p> <p>Seat Height Options: 17.75", 19.75"</p> <ul style="list-style-type: none"> • 70 degree and elevating leg rests • Solid and full poly tires • Flip back, height adjustable armrests • Fixed axle • Anti-tippers • Brake extensions 	<p>Breezy Easy Care 4000</p> <p>Weight Capacity: 250lbs</p>

MANITOBA WHEELCHAIR PROGRAM – QUICK REFERENCE GUIDE

MWP CATEGORY	TYPE OF USER / INTENDED USE	WHEELCHAIR FEATURES	WHEELCHAIR MODEL
<h2>Category 2C / 2C HD</h2>	<ul style="list-style-type: none"> • Full time use (min 6 hours/ day) • Independent, self-propeller indoors and outdoors • Physical or Functional limitations preclude the use of a Category 2A • Ambulation tolerance is limited (up to 25 meters with or without a gait aid) • Requires minimal horizontal axle adjustability (2" adult chair, 3" pediatric chair) • Seating and positioning interventions required to ensure functional use of mobility base <p>2C (HD) only: Client's weight exceeds the weight capacity of a Category 2A wheelchair</p>	<p>Seat Width Options: 14", 16", 18", 20", 22"</p> <p>Seat Depth Options: 16", 18", 20"</p> <p>Seat Height Options: 12.25" to 20.5" <i>(lower heights only achievable with smaller rear wheels)</i></p> <ul style="list-style-type: none"> • 60°, 70° and elevating leg rests • 22" and 24" rear wheel • Flip-back or T-style armrests • 2"-3" horizontal axle adjustability • Quick release axle • Angle adjustable back canes available • Anti-tippers • Brake extensions <p><i>(see spec sheets for full list of options)</i></p>	<p>Zippie GS (Pediatric) Weight Capacity: 165lbs</p> <p>Breezy Rubix* Weight Capacity 275lbs</p> <p>Breezy Rubix HD* Weight Capacity 352lbs</p> <p>Move Weight Capacity 265 lbs.</p> <p>Move HD Weight Capacity 350 lbs.</p> <p><small>*Models marked with * will be phased out effective April 2019. Chairs will be assigned at the discretion of the MWP as inventory allows</small></p>
<h2>Category 3</h2>	<ul style="list-style-type: none"> • Full time user (6+ hours per day) • Non-ambulatory • Medically/ functionally compromised and supported in the community with ++Caregiver support • Accessible home and entrance • Requires comprehensive postural evaluation along with seating and positioning interventions to ensure functional use of mobility base <p>(See Operations Manual for eligibility criteria)</p>	<p><u>Zippie TS</u> Seat Width: 10" – 18" Seat Depth: 13" – 20" Tilt Range: 45°</p> <p><u>Zippie Iris</u> Seat Width: 10" – 18" Seat Depth: 10" – 18" <i>(see Iris tilt range and other features below)</i></p> <p><u>Quickie SR45</u> Seat Width: 14" – 20" Seat Depth: 14" – 22" Seat Height: 16" – 19.5" Tilt Range: 45°</p> <p><u>Quickie Iris</u> Seat Width: 14" - 22" Seat Depth: 14" – 22"</p> <p>Tilt Range: up to 55°</p> <ul style="list-style-type: none"> • *Contracture Footrest System (J) • *Reclining Backrest (J) • *Dynamic Backrest (OTP) <p><i>(see spec sheets for full list of options)</i></p>	<p>Zippie TS Weight Capacity 165lbs</p> <p>Zippie Iris Weight Capacity 165lbs</p> <p>Quickie SR45 Weight Capacity 265lbs</p> <p>Quickie Iris Weight Capacity 265lbs (Std) Weight Capacity 350 lbs. (HD)</p>

MANITOBA WHEELCHAIR PROGRAM – QUICK REFERENCE GUIDE

MWP CATEGORY	TYPE OF USER / INTENDED USE	WHEELCHAIR FEATURES	WHEELCHAIR MODEL
<h2 style="font-size: 2em;">Category 4</h2>	<ul style="list-style-type: none"> • Full time user (6+ hours per day) • Accessible home and entrance • Independent with propulsion in all environments • Active user who accesses multiple environments daily • Requires ++adjustability in the configuration of the chair frame to address posture, positioning and sitting balance for mobility. • Requires >2"-3" horizontal axle adjustability to ensure self propulsion • Functional limitations preclude use of a Category 2C wheelchair • Requires comprehensive postural evaluation along with seating and positioning interventions to ensure functional use of mobility base <p style="text-align: center;">(See Operations Manual for eligibility criteria)</p>	<p>(See Spec Sheets for greater detail related to chair features)</p> <p>General Features</p> <ul style="list-style-type: none"> • Highly adjustable wheelchairs (independent front and rear seat to floor heights) • Up to 6" horizontal axle adjustability – <i>varies by model</i> • Low seat to floor height options: QXi, Helio A6, Helio Kids • Rigid and swing-away front rigging options (box frame only) • Standard and angle adjustable footplates • Variety of armrest styles • 3" of variability between front and rear seat to floor heights (e.g. dump) – <i>varies by model</i> • Angle adjustable back canes available • Cantilever and Box Frames are not often recycled, ordered to spec – <i>contact MWP Clinical Specialist or the manufacturer for support in filling out the spec sheet.</i> 	<p><u>Folding Frame:</u> <i>Sunrise Medical</i> Quickie: LXI*, QXi* Weight Capacity: 265lbs</p> <p>Quickie: Q2* Weight Capacity: 265lbs (std) 350lbs (HD)</p> <p>Quickie: M6 Weight Capacity: 650lbs</p> <p><i>Motion Composites:</i> Helio C2 Kids Weight Capacity: 175lbs</p> <p>Helio A6 Weight Capacity: 265lbs (std), 350lbs (HD)</p> <p><u>Box Frame:</u> <i>Sunrise Medical:</i> Quickie GP/ GPV/ GP Swing-away Weight Capacity: 265lbs</p> <p><u>Cantilever Frame:</u> <i>Sunrise Medical:</i> Q7* Weight Capacity: 265lbs</p> <p><i>Motion Composites:</i> Helio Apex Weight Capacity: 265lbs</p> <p style="font-size: 0.8em;">*Models marked with * will be phased out effective April 2019. Chairs will be assigned at the discretion of the MWP as inventory allows</p>

MANITOBA WHEELCHAIR PROGRAM – QUICK REFERENCE GUIDE

MWP CATEGORY	TYPE OF USER / INTENDED USE	WHEELCHAIR FEATURES	WHEELCHAIR MODEL
<h2 style="margin: 0;">Category 5 (Power)</h2>	<ul style="list-style-type: none"> • Full time user (6+ hours/day) • Unable to meet functional mobility needs via ambulation • Unable to meet functional mobility needs with use of a manual wheelchair • Intends to use power chair inside home and in the community • Deemed to safely operate a power wheelchair • Accessible home and entrance • Safe storage • Capable of maintaining chair/batteries with or without caregiver assistance • Requires comprehensive postural evaluation along with seating and positioning interventions to ensure functional use of mobility base <p>(See Operations Manual for eligibility criteria)</p>	<p>General Features</p> <ul style="list-style-type: none"> • Mid-wheel and rear wheel • Pediatric (Zippie) and adult (Quickie) platforms • Seat height range: 15.5” – 20.25” (without power positioning device) • Seat frame angles from 0° - 9° in 3° increments – <i>some restrictions apply, refer to spec sheet for more detail.</i> • Single and Multiple Drive controllers/joysticks • Swing Away joystick mount (OTP) • Centre-mount and swing-away front rigging options • Standard and angle adjustable footplates • Variety of armrest styles • Angle adjustable back canes available • Power Tilt (J), Recline (J), Elevating Leg rests (J) and Elevating Seat (OTP) available • MWP provides one drive control per client. • Specialty drive controls available with justification and completed driving assessment report – <i>contact the MWP Clinical Specialist for more information.</i> 	<p><u>MID-WHEEL DRIVE</u></p> <p>Zippie Xperience2 Weight Capacity: 300lbs</p> <p>Quickie Xperience2 Weight Capacity: 300lbs</p> <p>Quickie XperienceHD Weight Capacity: 400lbs</p> <p><u>BARIATRIC MID-WHEEL DRIVE</u></p> <p>Quickie Xcel2 Weight Capacity: 550lbs</p> <p><u>REAR WHEEL DRIVE</u></p> <p>Zippie Xplore2 Weight Capacity: 300lbs</p> <p>Quickie Xplore2 Weight Capacity: 300lbs</p> <p>Quickie XploreHD Weight Capacity: 400lbs</p>